


Dräger DrugCheck® 3000 Drug Testing Device

Use the Dräger DrugCheck® 3000 to find out within minutes if a person recently consumed certain drugs. The compact and quick oral fluid based drug test yields reliable results affordably and easily. The device does not require electricity and can be used anywhere.

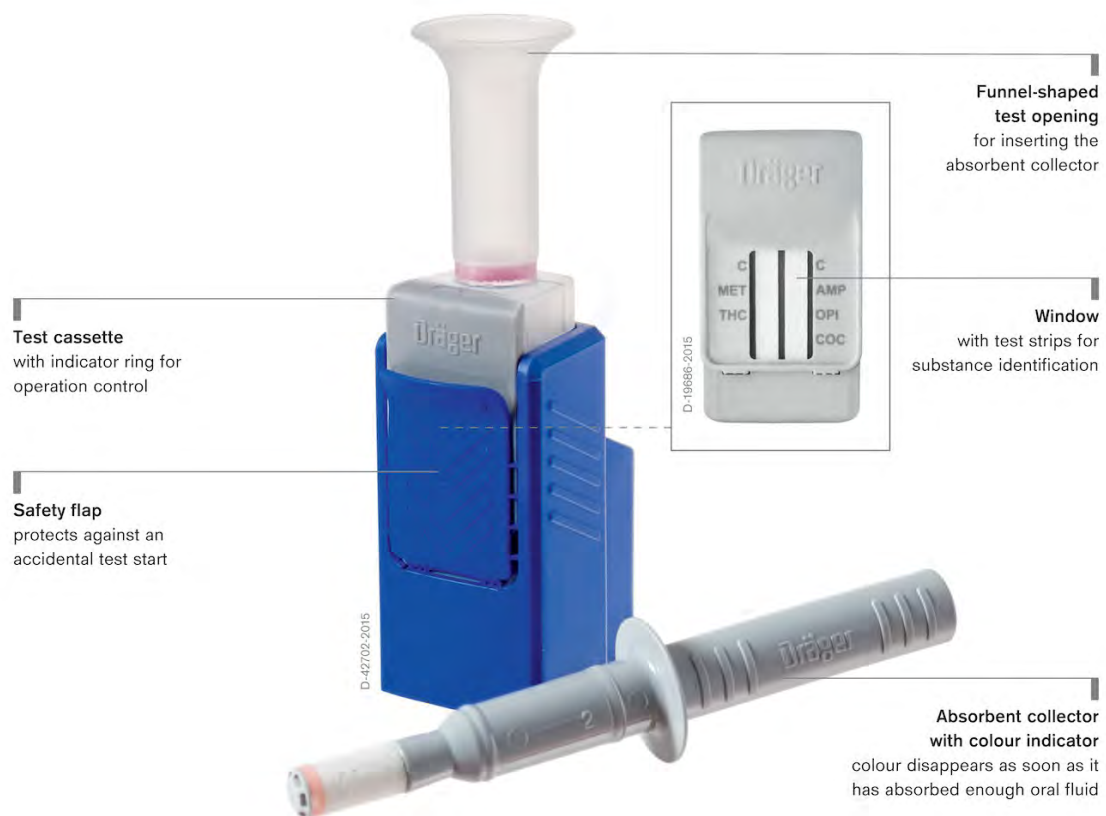

Benefits

Obtain samples easily and safely

The Dräger DrugCheck 3000 consists of two components: an absorbent collector for obtaining an oral fluid sample, and a test cassette for the analysis. The test cassette contains the buffer liquid and a window with two test strips, which display the control and test lines. The test itself is performed in three easy steps: swab the oral fluid sample, shake the test kit, wait for a brief incubation period, and then start the test. A colour indicator on the swab disappears as soon as it has absorbed enough oral fluid for a test.

As soon as the control lines appear in the window, you can read the results. If the test result is negative, a line will appear alongside the respective substance class (drug). This means that none of the target substance was detected in the sample. If a line fails to appear next to one of the substance classes, then the result for this substance is positive.

Fast and sensitive analysis

Check individuals for five substance classes simultaneously with Dräger DrugCheck 3000: cocaine, opiates, amphetamine, methamphetamines/designer drugs (e.g. Ecstasy) and cannabis (THC). Of all the substance classes listed, cannabis is the drug consumed most frequently and also the most difficult of all compounds to identify (THC). This is why the Dräger DrugCheck 3000 was optimised to detect THC, and now offers two measurement options: depending on the chosen waiting time (pre-incubation) of the test there is a fast option with a higher or sensitive option with a low THC cut-off. The entire test only takes 5 – 7 minutes.

Unambiguous on-the-spot drug detection

Its compact pocket sized design makes the DrugCheck 3000 easy to transport. It can be made ready for use quickly and easily, and is suitable for flexible use applications such as roadside checks and testing in safety-relevant workplaces*. The test kit has no electrical parts, which makes it safe to use in hazardous areas.

Additional benefits

Dräger has decades of experience in alcohol measuring equipment and drug detection methods. For the DrugCheck 3000, Dräger employed the testing principle of the proven Dräger DrugTest® 5000 system, which reliably detects even minute traces of THC. The disposable DrugCheck 3000 test kit cannot be manipulated and is hygienic to use.

*The DrugCheck® 3000 is sold only for law enforcement purposes in the USA.

Related Products



D-54720-2012

Dräger DrugTest® 5000

No pipetting, no drips, no timing: Collecting an oral fluid sample is quick and easy with the Dräger DrugTest® 5000. The collected sample can then be analysed immediately for accurate results on the spot.



D-6296-2009

Dräger SSK 5000

Ideal for surface drug testing applications, the Dräger SSK 5000 sampling system is quick and easy to use. When used with the Dräger DrugTest® 5000, the sample can be transferred to a test cassette and screened for possible drug contamination. The sample can also be stored and transported for further testing.



ST-13226-2007

Dräger DCD 5000

Simple, fast and secure detection of intoxicants and drugs: The Dräger DCD 5000 does not only enable an easy collection of saliva, you can also store the samples and send them to the laboratory for analysis – without the sample being falsified along the way.



D-4347-2014

Dräger SCK

To follow the chain of custody please use the confirmation kit with oral fluid collector for subsequent sampling for laboratory analysis of one sample including shipping material.

Technical Data

Test for detecting drugs in oral fluid

Dimensions (W x H x D)	32 x 111 x 57 mm 1.26 x 4.37 x 2.24 inches
Weight	< 30 g (0,07 lb)
Operating range	Operation: 5 °C to 30 °C (41 °F to 86 °F) at 5 % to 95 % relative humidity
Storage / transportation	4 °C to 25 °C (39 °F to 77 °F)
Time of measurement	Fast measurement: analysis < 5 minutes Sensitive measurement: analysis < 7 minutes
Selecting the measurement mode	Depending on the selected pre-incubation time and the desired THC cut-off fast or more sensitive reading.
Licenses / standards	Licensed as a medical product within the EU in accordance with Directive 98/79/EC on in vitro diagnostic medical devices (IVD). Outside of the European Union for non-medical use (Non-IVD) or for forensic use (only USA).

Ordering Information

Dräger DrugCheck® 3000	Description	Ordering no.
Test for detecting drugs in oral fluid		
Dräger DrugCheck® 3000 STK 5 IVD	Packaging unit 20 kits	83 25 500
Dräger DrugCheck® 3000 STK 5 Non-IVD	Packaging unit 20 kits	83 25 580

The Dräger DrugCheck® 3000 detects the following substances:

- Cocaine (COC)
- Opiates (OPI)
- Cannabis (THC)
- Amphetamines (AMP)
- Methamphetamines (MET)

CORPORATE HEADQUARTERS
Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany
www.draeger.com

**REGION EUROPE CENTRAL
AND EUROPE NORTH**
Dräger Safety AG & Co. KGaA
Revalstraße 1
23560 Lübeck, Germany
Tel +49 451 882 0
Fax +49 451 882 2080
info@draeger.com

REGION EUROPE SOUTH
Dräger Safety France SAS
25, rue Georges Besse
92182 Antony Cedex
Tel +33 1 46 11 56 00
Fax +33 1 40 96 97 20
europe.south@draeger.com

REGION MIDDLE EAST, AFRICA
Dräger Safety AG & Co. KGaA
Branch Office
P.O. Box 505108
Dubai, United Arab Emirates
Tel +971 4 4294 600
Fax +971 4 4294 699
contactuae@draeger.com

REGION ASIA PACIFIC
Draeger Safety Asia Pte Ltd
25 International Business Park
#04-20/21 German Centre
Singapore 609916
Tel +65 6308 9400
Fax +65 6308 9401
asia.pacific@draeger.com

**REGION CENTRAL
AND SOUTH AMERICA**
Dräger Panama S. de R.L.
Complejo Business Park,
V tower, 10th floor
Panama City
Tel +507 377 9100
Fax +507 377 9130
contactcsa@draeger.com

Locate your Regional Sales
Representative at:
www.draeger.com/contact

